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*Upper arm blood pressure monitor***REF: eta 629790000**

INSTRUCTIONS FOR USE

Dear customer, thank you for purchasing our product. Please read the operating instructions carefully before putting the appliance into operation and keep these instructions including the receipt and, if possible, the box with the internal packing.

I. SAFETY WARNING

- Consider the instructions for use as a part of the appliance and pass them on to any other user of the appliance.
- **The appliance is intended for use in households only and for similar use! It is not intended for use in health-care facilities and for commercial use!**
The values measured are intended for your information and monitoring the trends only and must not in any way replace a visit to the physician and having the blood pressure checked under qualified supervision. Proper blood pressure measurement must be performed by a qualified person (physician, nurse etc.) and in the context of an overall health condition of the person.
- **Do not use the device in moving vehicles (e.g. in a car).**
- **This appliance must not be used by children. This appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision. Children younger than 8 years must be kept out of reach of the appliance and its power cord.**

- To power the appliance, only apply safe small voltage in accordance with the type plate.

WARNING

- To power the appliance, only use a source of safe low voltage of 5V which complies with applicable standards (mobile phone chargers with a micro USB connector with double insulation are suitable).
- Do not use USB ports on other devices (e.g. computers, laptops, tablets, mobile phones, etc.) to power the appliance. Such devices may not be dimensioned for the current consumption of the appliance and it could damage them.

- Always unplug the appliance from charger if you leave it unattended and cleaning.
- The appliance has no charging source of its own, it is only designed for charging via USB (an adapter with USB output with output voltage of 5.0 V, and output current of at least 1000 mA). Before making any connection, please pay attention to instructions of the manufacturer of the respective power source.
- Never use the appliance if the adapter is damaged, if the appliance does not operate properly, fell on the ground and

suffered any damage, or fell into water. In such case bring the appliance to the specialised service centre to verify the appliance safety and proper operation.

- The appliance and the connected adapter must not be used at places where it could fall into bath tub, wash basin or swimming pool. If the appliance fell into water anyway, do not take it out! First of all, unplug the adapter from the socket and only then take the appliance out. Then bring the appliance to a specialised service centre to check its safety and proper functionality.

- The mains adapter supplied with the appliance is only designated for this appliance.
- Never plug the adapter in the electrical socket or unplug it with wet hands.
- **Never immerse the adapter into water or other liquids (not even partly) and do not wash it in running water!**
- The appliance gets hot when charging, which is absolutely normal.
- Charge the battery at a temperature close to the room temperature.
- Never charge the appliance in direct sunlight or close to heat sources, and if the temperature is below 10 °C or above 40 °C, adversely affects the accumulator service life.
- Prevent hanging the power adapter cord loosely over desktop edge where children may reach it.
- The power cord of the adapter must not be damaged with sharp or hot objects or open fire, and must not be immersed into water or bent over sharp edges.
- All text in other languages and pictures shown on the packaging, or the product itself, are translated and explained at the end of this language mutation.
- **WARNING:** There is a risk of injury in case of incorrect use (not in accordance with the manual) of the appliance.

- The producer is not responsible for damage caused by improper use of the appliance and its guarantee does not cover the appliance in the case of failure to comply with the safety warnings.
- This device is intended for adult use in homes only.
- The device is not suitable for use on neonatal patients, pregnant women, patients with implanted, electronic devices, patients with pre-eclampsia, premature ventricular beats, atrial fibrillation, peripheral, arterial disease and patients undergoing intravascular therapy or arterio-venous shunt or people who received a mastectomy. Please consult your doctor prior to using the unit if you suffer from illnesses.
- The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- The device is not intended for patient transport outside a healthcare facility.
- The device is not intended for public use.
- This device is intended for non-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on extremities other than the arm or for functions other than obtaining a blood pressure measurement.
- Do not confuse self-monitoring with self-diagnosis. This unit allows you to monitor your blood pressure. Do not begin or end medical treatment without asking a physician for treatment advice.
- If you are taking medication, consult your physician to determine the most appropriate time to measure your blood pressure. Never change a prescribed medication without consulting your physician.
- Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor. Consult your doctor if you have any question about your blood pressure.
- When using this device, please pay attention to the following situation which may interrupt blood flow and influence blood circulation of the patient, thus cause harmful injury to the patient: consecutive multiple measurements; the application of the cuff and its pressurization on any arm where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present; inflating the cuff on the side of a mastectomy.
- **Warning:** Do not apply the cuff over a wound; otherwise it can cause further injury.
- Do not inflate the cuff on the same limb which other monitoring ME equipment is applied around simultaneously, because this could cause temporary loss of function of those simultaneously-used monitoring ME equipment.
- On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately. Prolonged high pressure applied to the arm may lead to an ecchymosis.
- Please check that operation of the device does not result in prolonged impairment of patient blood circulation.
- When measurement, please avoid compression or restriction of the connection tubing.
- The device cannot be used with HF surgical equipment at the same time.
- This device is contraindicated for any female who may be suspected of, or is pregnant. Besides providing inaccurate readings, the effects of this device on the fetus are unknown.
- Too frequent and consecutive measurements could cause disturbances in blood circulation and injuries.
- This unit is not suitable for continuous monitoring during medical emergencies or operations. Otherwise, the patient's arm and fingers will become anaesthetic, swollen and even purple due to a lack of blood.
- When not in use, store the device in a dry room and protect it against extreme moisture, heat, lint, dust and direct sunlight. Never place any heavy objects on the storage case.

- **This device may be used only for the purpose described in this booklet. The manufacturer cannot be held liable for damage caused by incorrect application.**
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in this booklet.
- The patient is an intended operator.
- To avoid measurement errors, please avoid the condition of strong electromagnetic field radiated interference signal or electrical fast transient/burst signal.
- The blood pressure monitor and the cuff are suitable for use within the patient environment. If you are allergic to polyester, nylon or plastic, please don't use this device.
- If you experience discomfort during a measurement, such as pain in the arm or other complaints, press the **START/STOP** button to release the air immediately from the cuff. Loosen the cuff and remove it from your arm.
- If the cuff pressure reaches 30,59 kPa (230 mmHg), the unit will automatically deflate. Should the cuff not deflate when pressures reaches 30,59 kPa (230 mmHg), detach the cuff from the arm and press the **START/STOP** button to stop inflation.
- Before use, make sure the device functions safely and is in proper working condition. Check the device, do not use the device if it is damaged in any way. The continuous use of a damaged unit may cause injury, improper results, or serious danger.
- Do not wash the cuff in a washing machine or dishwasher!
- The service life of the cuff may vary by the frequency of washing, skin condition, and storage state. The typical service life is 5000 times.
- Don't open or repair the device by yourself in the event of malfunctions. The device must only be serviced, repaired and opened by individuals at authorized sales/service centers.
- Keep the unit out of reach of infants, young children or pets to avoid inhalation or swallowing of small parts. It is dangerous or even fatal.
- At least 30 min required for ME equipment to warm from the minimum storage temperature between uses until it is ready for intended use. At least 30 min required for ME equipment to cool from the maximum storage temperature between uses until it is ready for intended use.
- Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect this equipment and should be kept at least a distance away from the equipment (min. 30 cm).
- The maximum temperature that the attached part can reach is 41.8 °C, while the ambient temperature is around 40 °C and the contact time of the patient with the cuff should be less than 10 minutes.
- Use only accessories approved by the manufacturer. Use of unapproved accessories may cause damage to the device and injury to the user.

Wireless connectivity and compatibility

- HP TRONIC Zlín, spol. s r.o. can in no way be held liable for any accidental, indirect or subsequent damage, data loss or any other loss incurred as a result of information leakage during communication through BLUETOOTH, Wifi technology.
- 100% successful wireless communication cannot be guaranteed with all Smart devices. However, given the variety of devices available in the market, even if a device is in compliance with the specifications, there might be cases when the character or specifications of the given Smart device do not allow connectivity, or hinder connectivity in any way. This is not a reason to make a warranty claim.

- Radio waves transmitted by devices may interfere with operation of medical equipment. Such interference may cause a failure. Therefore, do not use the device in the following places:
 - in hospitals, trains, planes, petrol stations and in areas where flammable gases may be present.
 - in the vicinity of automatic doors or fire detectors.

The App and the web platform is owned and operated by a third entity (hereinafter referred to as the “Entity”) other than company HP TRONIC Zlín, s r.o. (hereinafter referred to as the “Seller”). Personal data are not transferred between the Entity and the Seller. When signing in to the App, the customer submits their personal data directly to the Entity, which becomes the personal data administrator. The App and related personal data protection are governed by the rules of the Entity.

HP TRONIC Zlín, spol. s r.o. uses the services of MedM Inc., which offers its professional multiplatform applications for manufacturers of various medical and measuring devices for home use. In addition to the applications, MedM Inc. also offers a web platform (<https://health.medm.com/>), where you can manage your measurements more easily and clearly after logging in (with the same username and password as the application). MedM Inc. regularly updates the application, but HP TRONIC Zlín, spol. s r.o. has only limited possibilities to modify it.

Development and any changes to the application are made exclusively by the owner of the application, i.e. MedM Inc. The company HP TRONIC Zlín, spol. s r.o. has no access to any data related to the use of the application.

The basic features of the app and platform are free, or you can take advantage of MedM Inc.'s offer to purchase a subscription that extends the features and capabilities of both the app and the web portal. For more information on this option, please visit www.etasince1943.com/care.

The web platform available at <https://health.medm.com/> is only a supplementary service and a possibility to extend the use of the product. It is not a part or accessory of the ETA629790000 pressure gauge! At the same time, HP TRONIC Zlín, spol. s r.o. does not guarantee the functionality of this platform or its localization into the customer's official language.

II. GENERAL INFORMATION

You have bought a device, at a first glance looking and acting as an ordinary pressure gauge. However, if connected via bluetooth to any Smart device with Android or iOS operating system with **MedM BP (Blood Pressure Diary by MedM)** App installed, the pressure gauge becomes a multifunctional modern assistant to help monitor and analyse the pressure and heart rate.

After linking the pressure gauge with a Smart device, the measurement results are transferred via bluetooth to the **MedM BP (Blood Pressure Diary by MedM)** App.

All other information, operating instructions of the app, instructions and hints for successful pairing with home network can be found at:

www.etasince1943.com/care

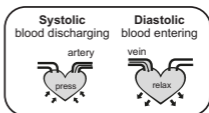
The app is available from Apple Store or Google Play database,

Measurement Principle

This product uses the Oscillometric Measuring method to detect blood pressure. Before every measurement, the unit establishes a "zero pressure" equivalent to the atmospheric pressure. Then it starts inflating the arm cuff, meanwhile, the unit detects pressure oscillations generated by beat-to-beat pulsatile, which is used to determine the systolic and diastolic pressure, and also pulse rate.


What are systolic pressure and diastolic pressure?

When ventricles contract and pump blood out of the heart, the blood pressure reaches its maximum value in the cycle, which is called systolic pressure. When the ventricles relax, the blood pressure reaches its minimum value in the cycle, which is called diastolic pressure.

What is the standard blood pressure classification?

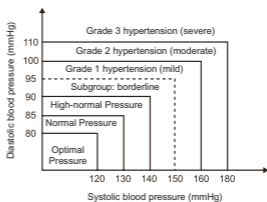The blood pressure classification published by World Health Organization (WHO) and International Society of Hypertension (ISH) in 1999 is as follows:

| Blood Pressure (mm Hg) | Optimal | Normal | High-normal | Mild | Moderate | Severe |
|------------------------|---------|---------|-------------|---------|----------|--------|
| SYS | <120 | 120-129 | 130-139 | 140-159 | 160-179 | ≥180 |
| DIA | <80 | 80-84 | 85-89 | 90-99 | 100-109 | ≥110 |

CAUTION

Only a physician can tell your normal BP range. Please contact a physician if your measuring result falls out of the range. Please note that only a physician can tell whether your blood pressure value has reached a dangerous point.

Irregular Heartbeat Detector (IHB)

An irregular heartbeat is detected when a heartbeat rhythm varies while the device is measuring systolic pressure and diastolic pressure. During each measurement, blood pressure monitor will keep a record of all the pulse intervals and calculate the average value of them. If there are two or more pulse intervals, the difference between each interval and the average is more than the average value of $\pm 25\%$, or there are four or more pulse intervals, the difference between each interval and the average is more than the average value of $\pm 15\%$, then the irregular heartbeat symbol will appear on the display with the measurement result.

CAUTION

The appearance of the IHB icon indicates that a pulse irregularity consistent with an irregular heart-beat was detected during measurement. Usually this is NOT a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

Note

ETA a.s. is not a licensed provider of health and cosmetic care and does not have expertise in the field of treatment, diagnostics of the health condition and related effects or suitability for the use of appliances that may affect the health condition in any way. By using this appliance, you agree that you use it at your own risk and assume all risks of injury or deterioration. You also agree to waive any claims (whether known or unknown) against ETA a.s. arising from improper use of this appliance or through negligence. Before using any appliances that may affect your health, first consult your doctor about the suitability of their use.

III. PREPARATION FOR USE

Remove all the packing material and remove the pressure gauge.
Remove adhesive film, stickers, or paper.

First use - activating the pressure gauge

After purchasing a blood pressure device, you must first activate it. Long press the "START/STOP" button for about 5 seconds to activate the device.

Charging the battery

– Standard charging time is approximately 2.5 - 3 hours and a fully charged new battery will provide up to 4 hours of operation (up to 150 measurements). The operating time decreases depending on the load on the appliance as well as aging and wear of the battery.

- 1) Make sure the appliance is switched off.
- 2) Plug the included USB cable into the charging connector **A3** of the appliance and connect the other end to the source; eg a compatible AC mains adapter or to a backup power source. The appliance cannot be used while charging.
- 3) Do not exceed the specified charging time!
- 4) Charging is indicated by the **A4** light (blue). When the battery is fully charged the **A4** signal goes out.

Comment

- Do not leave the USB cable connected in the appliance.
- The device heats up during charging or operation, which is normal and is not a reason for a complaint.
- This appliance contains batteries that are not replaceable.

Descriptions (obr.1)

A – monitor

- A1 – START/STOP button
- A2 – LED display
- A3 – charging connector
- A4 – light signalling

B – cuff

C – USB-C cable *

* USB-C cable: you can connect a source with USB to this USB-C cable (e.g. AC power adapter, Input: 100-240 V, 50-60 Hz, 0.2A max, Output: 5 V $\overline{\text{---}}$ 1000 mA).

DISPLAY DESCRIPTION

| SYMBOL | DESCRIPTION | EXPLANATION |
|---|--------------------------|------------------------------------|
| SYS | Systolic blood pressure | High blood pressure |
| DIA | Diastolic blood pressure | Low blood pressure |
| PULSE /min | Pulse display | Pulse in beats per minute |
|  | Pulse | Pulse detection during measurement |

| SYMBOL | DESCRIPTION | EXPLANATION |
|---|--|---|
| mmHg | mmHg | Measurement Unit of the blood pressure |
|  | Battery indicator | Current battery level indicator +"bAt Lo" = low battery |
|  | Irregular heart-beat | Blood pressure monitor is detecting an irregular heartbeat during measurement. |
| light signalling A4 | Blood pressure and operation indicator | Green – normal pressure Yellow – high pressure Blue - charging (goes off when charged) |
|  | Cuff condition | Cuff use indicator. The icon lights up if it is properly attached. The icon flashes if it is insufficiently fixed. Pokud nepřestane po chvíli blikat, přerušte měření a znovu manžetu upevněte. |
| U1, U2 | User 1 / User 2 | Measurement for User 1 / User 2 |
|  | Bluetooth icon | This icon flashes when a Bluetooth connection is established, and glows when connected. |
|  | Motion indicator | Motion may result in an inaccurate measurement |
|  | Sending | Indicates that data is being sent. |

IV. INSTRUCTIONS FOR USE

A) TIE THE CUFF (fig. 2, 3 a 4)

- Remove all jewelry, such as watches and bracelets from your left arm.
Note: If your doctor has diagnosed you with poor circulation in your left arm, use your right arm.
- Roll or push up your sleeve to expose the skin. Make sure your sleeve is not too tight.
- Hold your arm with your palm facing up and tie the cuff on your upper arm. The blood pressure monitor should be placed on the inside of the arm.
- The cuff should be snug but not too tight. You should be able to insert one finger between the cuff and your arm.
- Sit comfortably with your tested arm resting on a flat surface. Place your elbow on a table so that the cuff is at the same level as your heart. Turn your palm upwards.

Sit upright in a chair, and take 5-6 deep breaths. Fig.4

- Helpful tips for Patients, especially for Patients with Hypertension:

– Rest for 5 minutes before first measuring.

- Wait at least 3 minutes between measurements. This allows your blood circulation to recover.
- Take the measurement in a silent room.
- The patient must relax as much as possible and do not move and talk during the measurement procedure.
- The cuff should maintain at the same level as the right atrium of the heart.
- Please sit comfortably. Do not cross your legs and keep your feet flat on the ground.
- Keep your back against the backrest of the chair.

For a meaningful comparison, try to measure under similar conditions.

For example, take daily measurements at approximately the same time, on the same arm, or as directed by a physician.

B) USER SELECTION AND MEASUREMENT

- 1) Press the **A1 START/STOP** button on the switched off pressure gauge briefly to activate the user setup mode - **U1** or **U2** representing the user number will start flashing. If you immediately press **A1 START/ STOP** again, the user is switched: user **U1** > user **U2** > **U1**....
- 2) If no action is taken within 2 seconds, the actual measurement starts.
- 3) If the measurement is successful, the results will appear on the A2 display for a few moments.
- 4) Press the "**A1 START/ STOP**" button to end the measurement sooner (switching off). Otherwise, the device will switch itself off after displaying the values.

Notes

- For each user, the pressure gauge has an internal memory of 199 measurements.
- Without using the app, the pressure gauge will only display the results of the current measurement. Previous measurements are stored in its memory and will be accessible when connected to the mobile app.
- After successful pairing with the app, the date and time are automatically synchronized for correct matching of results.
- If there is untransmitted data in the app, the  symbol will appear on the display at the start of the measurement.
- If the measurement results display a motion and irregular heartbeat indicator icon, the measurement may be biased.

C) LINKING WITH A SMART DEVICE

Long press the **A1** button to put the pressure gauge into pairing mode (the icon  starts flashing).

The process of pairing the appliance with the application is described and shown in the instructions available at www.etasince1943.com/care

V. TROUBLESHOOTING OF POSSIBLE PROBLEMS

| PROBLEM | SYMPTOM | CHECK IT OUT | SOLUTION |
|---------------------|-------------------------------|--|---|
| It doesn't work | The display does not light up | The battery is discharged. | Recharge the battery. |
| It does not measure | | Be calm during the measurement and place the cuff correctly. | Make sure you follow the instructions when measuring. |
| Air leak | Air is leaking | Check the cuff for damage | If the cuff is damaged, contact the service center. |

If the measurement is not performed correctly, the following error messages may appear. Make sure that you follow the instructions in the manual when measuring and repeat the measurement if necessary.

| Error messages | Cause |
|---|---|
| E 1 | incorrectly positioned cuff |
| E 2 /  | excessive movement detected during measurement, or pulse too weak |
| E 3 | pulse not detected during measurement |
| E 4 | measurement failed |
| E xx | calibration error (x may represent a digit) |
| out | the measured values are outside the available range |

VI. MAINTENANCE

Dust environment may affect the performance of the unit. Please use the soft cloth to clean the whole unit before and after use. Don't use any abrasive or volatile cleaners. In order to get the best performance, please follow the instructions below.

1. Cleaning Process:

Step 1: Make sure to switch off the device prior to cleaning.

Step 2: Use a soft cloth wetted with soapy water to clean the cuff first, and then use a soft cloth wetted with clear water to remove residual soap until there is no visible residual contaminants. Attention shall be paid to avoid liquid invasion into the cuff.

Step 3: Use a dry soft cloth to wipe the cuff, in order to remove residual moisture.

Step 4: Dry the cuff at a well-ventilated place after cleaning.

2. Disinfection Process:

Step 1: Make sure to switch off the device prior to disinfection.

Step 2: Use a soft cloth wetted with 70% isopropanol to disinfect the cuff for about 10 minutes. Attention shall be paid to avoid liquid invasion into the cuff.

Step 3: Use a clean dry cloth or towel to wipe off the disinfectant until there is no visible residue.

Step 4: Dry the cuff at a well-ventilated place after disinfection.

Suggestion: Frequency of Cleaning and Disinfection: For single patient multiple use, it's recommended to clean the device surface once a month or whenever it's necessary. For multiple patient multiple use, it's recommended to clean the device every time before and after usage. Maintenance procedures shall be taken as per instruction.

The manufacturer reserves the right to change the technical specifications for the respective model. The manufacturer reserves the right to change/ update the MedM BP (Blood Pressure Diary by MedM) application without prior notice. Due to the character of such changes/updates, some information or procedures in this manual might become obsolete or might be missing.

The capacity of the product battery decreases over time. Decreasing of battery capacity occurs during normal use of the appliance and is considered usual wear and tear of the appliance. Decreasing battery capacity is not a defect of the appliance.

More extensive maintenance or maintenance requiring intervention in the inner parts of the appliance must be carried out by professional service! Failure to comply with the instructions of the manufacturer will lead to expiration of the right to repair!

The information regarding warranty and product repairs you can find on the website www.eta.cz/servis-eu. **Hereby, ETA a.s. declares that the radio equipment type ETA629790000 is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at www.eta.cz/declaration-of-conformity.**

VII. ENVIRONMENTAL PROTECTION

If the dimensions allow, there are labels of materials used for production of packing, components and accessories as well as their recycling on all parts. The symbols specified on the product or in the accompanying documentation mean that the used electrical or electronic products must not be disposed of together with municipal waste. For proper disposal, hand them over at special collection places where they will be accepted free of charge. Suitable disposal of the product can help to maintain valuable natural resources and to prevent possible negative impacts on the environment and human health, which could be the possible consequences of improper waste disposal. Ask for more details at the local authorities or at a collection site. Fines can be imposed for improper disposal of this type of waste in agreement with the national regulations. Dispose of the battery safely by means of collection networks intended for this purpose. Never dispose of the batteries by burning!

Taking out the battery

– Do not attempt to replace your blood pressure monitor's battery. It is built-in and not changeable.

- Only charge the battery in accordance with the user instructions supplied with the blood pressure monitor.
- Avoid charging your blood pressure monitor in extremely high or low temperatures.
- Do not use your blood pressure monitor while you are charging it.
- Do not attempt to disassemble the blood pressure monitor or force open the built-in battery.
- Do not clean the blood pressure monitor when it is being charged. Always unplug the charger first before cleaning the blood pressure monitor.
- Do not dispose of your blood pressure monitor in a fire. The battery could explode causing injury or death.
- Batteries (battery pack or batteries installed) shall not be exposed to excessive heat such as sunshine, fire or the like. exposed to excessive heat such as sunshine, fire or the like.

VIII. TECHNICAL DATA

| Parameter | Description |
|---|---|
| Measurement mode | Oscillographic testing mode |
| Measurement range | Rated cuff pressure: 0 mm Hg ~ 299 mmHg Measurement pressure: SYS: 60 mm Hg ~ 230 mmHg DIA: 40 mmHg ~ 130 mmHg Pulse value: (40-199) beat/minute |
| Accuracy | Pressure: 5 °C - 40 °C within ± 0.4 kPa (3 mmHg) Pulse value: ± 5 % Clinical validation: ± 5 mmHg Standard deviation: ≤ 8 mmHg |
| Normal working condition | A temperature range of: +5 °C to +40 °C A relative humidity range of: 15 % to 90 % Non-condensing, but not requiring a water vapour partial pressure greater than 50 hPa; An atmospheric pressure range of: 700 hPa to 1060 hPa |
| Storage temperature | -20 °C ~ +60 °C, less than 93 % HR; non-condensing, at water vapour pressure up to 50 hPa, Atmospheric pressure ranging from 500 hPa to 1060 hPa |
| Measurement perimeter of the upper arm | About 22 cm - 42 cm |
| Protection against ingress of water | IP22 This means that the device is protected against objects with a diameter of more than 12 mm and against vertical, direct spraying of water up to an angle of 15°. |
| Power supply | 3,6 V Li-Ion; 1000 mAh |
| Net weight of device | 0,206 kg |
| Size of the product (mm) | 123 x 44 x 22 |

| Parameter | Description |
|-------------------------------|---|
| Degree of protection | Type BF applied part |
| Software version | A01 |
| Bluetooth RF frequency range: | 2400 MHz to 2480 MHz |
| Output power | 0 dBm |
| PER | <10% |
| Delay | 50 ms |
| Device classification | Battery power mode: internally powered ME device ; AC adapter power mode: Class II ME device |
| Transmitting Distance: | 10 meters |
| Types of use/reuse | Multiple patient multiple use |
| Expected Lifetime | Device (with the cuff): 3 years or 10000 times (may vary based on usage conditions) Battery: About 100 times after fully charged |

Essential performance: Accuracy of measuring blood pressure and pulse rate

| | |
|---------------------|---|
| Measurement Range | Systolic pressure: 60-230 mmHg Diastolic pressure: 40-130 mmHg Pulse: 40-199 beats/minute |
| Rated Cuff Pressure | 0-299 mmHg (0-39,9 kPa) |
| Accuracy | Pressure: ± 3 mmHg / 0,4 kPa; Pulse: $\pm 5\%$ |

The Basis Safety of the Blood Pressure Monitor (TMB-2296-B) is as following: Deviation from normal operation that poses an unacceptable risk to the patient or operator.

WARNINGS AND SYMBOLS USED ON THE APPLIANCE, PACKAGING OR IN THE INSTRUCTIONS MANUAL:

HOUSEHOLD USE ONLY. DO NOT IMMERSE IN WATER OR OTHER LIQUIDS. TO AVOID DANGER OF SUFFOCATION, KEEP THIS PLASTIC BAG AWAY FROM BABIES AND CHILDREN. DO NOT USE THIS BAG IN CRIBS, BEDS, CARRIAGES OR PLAYPENS. THIS BAG IS NOT A TOY.

Removable power supply unit

Indications for Use

- This Blood Pressure Monitor is a digital monitor intended for use in measuring blood pressure and pulse rate with arm circumference ranging from 22cm to 32cm (about 8¾"-12½") or 22cm to 42cm (about 8¾"-16½").
- It is intended for adult indoor use only.
- The battery of TMB-2296-B is built-in rechargeable lithium-ion battery, please do not disassemble it by the unauthorized maintenance personnel.
- If the battery cannot charge the power normally or the blood pressure monitor cannot use normally, please connect with the authorized maintenance personnel. If the battery is fully charged, it can be used for about 100 times.
- Store and use the blood pressure monitor at the cool, dry and ventilated environment. Avoid to approach to the fire and the heat source, or it will cause the battery explode.
- Only can use the authorized AC Adaptor (Model: BLJ06L050100U-V \ BLJ06L050100U-S \ BLJ06L050100U-B) to charge the power. You cannot use the blood pressure monitor during the process of charging.
- During the process of charging, the LED indicator of the START/STOP button will be lighted. When the charging is finished, please pull the plug in time.
- When charging, shall not touch charging connector and the patient simultaneously.

Device Classification

Battery Powered Mode: Internally Powered ME Equipment

AC Adaptor Powered Mode: Class II ME Equipment

Manufactured by: Guangdong Transtek Medical Electronics Co., Ltd.
Zone A, No.105 ,Dongli Road, Torch Development District
Zhongshan,528437,Guangdong,China

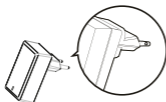
Authorized European Representative

MDSS GmbH; Schiffgraben 41, 30175 Hannover, Germany

Importer: ETA a.s.; Křižíkova 148/34; Praha 8 - Karlín, 186 00

♥ Authorized Component

Please use the authorized adapter. (Optional!)

Adapter

Type: BLJ06L050100U-V
BLJ06L050100U-S
BLJ06L050100U-B

Input: 100-240V, 50-60Hz, 0.2A max
Output: 5V \square 1000 mA

| | | | |
|---|---|---|---|
|  | Symbol for „MANUFACTURER“ |  | BF-type anti –shock degree for the application part |
|  | Date of manufacture |  | Direct current |
| | |  | Serial number |
|  | Recyclable |  | Medical equipment |
|  | For indoor use only |  | Appliance of protection class II |
|  | Consult the instructions for use | | |
|  | Warning information, refer to the attached document | | |
|  | The symbol indicates that the product must not be discarded as unsorted waste but must be taken to a separate collection facility for recovery and recycling. | | |
|  | The label means that the manufacturer has assessed the product and concluded that it meets EU safety, health, and environmental requirements. | | |
|  | Authorized representative in the European Community/European Union | | |

EMC Guidance

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Technical description:

- 1) All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the expected service life.
- 2) Guidance and manufacturer's declaration -electromagnetic emissions and Immunity

Table 1

| Guidance and manufacturer's declaration - electromagnetic emissions | |
|---|------------|
| Emissions test | Compliance |
| RF emissions CISPR 11 | Group 1 |
| RF emissions CISPR 11 | Class B |
| Harmonic emissions IEC 61000-3-2 | Class A |
| Voltage fluctuations/flicker emissions IEC 61000-3-3 | Comply |

Table 2

| Guidance and manufacturer's declaration – electromagnetic Immunity | | |
|---|---|---|
| Immunity Test | IEC 60601-1-2 Test level | Compliance level |
| Electrostatic discharge (ESD) IEC 61000-4-2 | ±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air | ±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air |
| Electrical fast transient/ burst IEC 61000-4-4 | ±2 kV for power supply lines 1 kV signal input/output 100 kHz repetition frequency | ±2 kV for power supply lines N/A 100 kHz repetition frequency |
| Surge IEC61000-4-5 | ±0.5 kV, ±1 kV differential mode 0.5 kV, ±1 kV,±2 kV common mode | ±0.5 kV, ±1 kV differential mode |
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 | 0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°.0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°.0 % UT; 250/300 cycle | 0 % UT; 0,5 cycle. At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°.0 % UT; 1 cycle and 70 % UT; 25/30 cycles; Single phase: at 0°.0 % UT; 250/300 cycle |
| Power frequency magnetic field IEC 61000-4-8 | 30 A/m 50Hz/60Hz | 30 A/m 50Hz/60Hz |
| Conducted RF IEC61000-4-6 | 3 V; 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz | 3 V; 0,15 MHz – 80 MHz 6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz |
| Radiated RF IEC61000- 4-3 | 10 V/m 80 MHz – 2,7 GHz; 80 % AM at 1 kHz | 10 V/m 80 MHz – 2,7 GHz; 80 % AM at 1 kHz |
| NOTE U_i is the a.c. mains voltage prior to application of the test level. | | |

Table 3

| Guidance and manufacturer's declaration - electromagnetic Immunity | | | | | | | |
|---|----------------------|------------|--|--------------------------------------|----------------|--------------|---------------------------|
| Radiated RF IEC61000-4-3; (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment) | Test Frequency (MHz) | Band (MHz) | Service | Modulation | Modulation (W) | Distance (m) | IMMUNITY TEST LEVEL (V/m) |
| | 385 | 380-390 | TETRA 400 | Pulse modulation b) 18Hz | 1,8 | 0,3 | 27 |
| | 450 | 430-470 | GMRS 460, FRS 460 | FM c) \pm 5kHz deviation 1kHz sine | 2 | 0,3 | 28 |
| | 710 | 704-787 | LTE Band 13,17 | Pulse modulation b) 217Hz | 0,2 | 0,3 | 9 |
| | 745 | | | | | | |
| | 780 | | | | | | |
| | 810 | 800-960 | GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5 | Pulse modulation b) 18Hz | 2 | 0,3 | 28 |
| | 870 | | | | | | |
| | 930 | | | | | | |
| | 1720 | 1700-1990 | GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4,25; UMTS | Pulse modulation b) 217Hz | 2 | 0,3 | 28 |
| | 1845 | | | | | | |
| | 1970 | | | | | | |
| | 2450 | 2400-2570 | Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7 | Pulse modulation 217 Hz | 2 | 0,3 | 28 |
| | 5240 | 5100-5800 | WLAN 802.11 a/n | Pulse modulation 217 Hz | 0,2 | 0,3 | 9 |
| 5500 | | | | | | | |
| 5785 | | | | | | | |

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© DATE 10/07/2024
Omega electric GmbH, Servicezentrale, Industriering 2, D- 04626 Schmölln.